seca

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CONTENTS

1. About this document	27	5. Before you get started	40
2. Description of device	27	5.1 Wall mounting of cabled remote display	40
2.1 Intended purpose	27	Installing the wall brackets . . .	40
2.2 Functional description	27	5.2 Establish power supply	41
2.3 User qualification	28	Connect power supply unit . .	41
Assembly	28	Charge battery	41
Operation	28	6. Hygienic reprocessing	41
3. Safety information	29	6.1 Cleaning	42
3.1 Safety information in this document	29	6.2 Desinfection	42
3.2 Basic safety precautions	29	6.3 Sterilisation	42
Handling the device	29	7. Functional check	43
Handling the chair scales	30	8. What do I do if	44
Handling the baby scales	31	9. Technical data	45
Preventing electric shock	31	10. Optional accessories	46
Avoiding infections	32	11. Optional accessories for seca 360° wireless network . . .	46
Avoiding injury	32	12. Spare parts	47
Avoiding damage of the device	32	13. Disposal	47
Handling the measurement results	33	13.1 Device	47
Handling the packaging material	34	13.2 Batteries (including rechargeables)	47
3.3 Handling the (rechargeable) batteries	34		
4. Overview	35		
4.1 Control elements	35		
4.2 Symbols in display	37		
4.3 Information on device and on rating plate	38		
4.4 Information on the packaging .	39		

1. ABOUT THIS DOCUMENT

This document is part of the product's user manual.

This document contains **general safety instructions and additional information** on the individual sections of the manual.

Keep this document in a safe place along with the user manual and ensure that both documents are available at all times.

2. DESCRIPTION OF DEVICE

2.1 Intended purpose

The medical scales and measuring stations from seca are mainly used in hospitals, doctors' surgeries and inpatient care facilities in accordance with national regulations.

The devices are used for conventional weight and size determination, determination of the general nutritional status, and help the treating physician to make a diagnosis or treatment decision.

In order to make a precise diagnosis, besides determination of the weight, further targeted examinations must be set up by the physician, and their results must be considered.

2.2 Functional description

Weight measurement with electric scales is done via 1 or 4 load cells dependent on the model; mechanical scales use a load lever system.

Dependent on the model, the following functions may be available on your scales:

- **hold:** Permanent display of the measurement result after the load has been removed from the scales.
- **tare:** Taring of additional weight, e.g. of a towel for the current weighing process.
- **pretare:** Permanent storage of a weight (e.g. blanket weight for clothing), which is deducted automatically in each weighing process
- **2 in 1:** Weight of a child can be determined with an adult standing on the scales with the child.

- **BMI:** Determination of body mass index through the measured weight value and the manually entered body size of the patient.
- **BMIF:** Calculation of the ingested quantity of food.
- **Limit value:** For stress exercises after e.g. knee surgery
- **Manual switchover of weighing range:** For multi-range scales, different weighing ranges can be selected. Weighing range 1 offers a more precise weight indication with a reduced load-bearing capacity. In the second weighing range, and, dependent on the model also in a third, you can use a larger loading capacity of the scales with an adjusted split of the weight display.

2.3 User qualification

Assembly

Devices that are shipped partially assembled may only be mounted by sufficiently qualified persons such as specialist dealers, hospital technicians or service technicians.

Operation

The device may only be operated by healthcare professionals.

3. SAFETY INFORMATION

3.1 Safety information in this document

DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

WARNING!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

CAUTION!

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

ATTENTION!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measured results may be incorrect.

NOTE:

Includes additional information about use of the device.

3.2 Basic safety precautions

Handling the device

- Please observe the information in this document and in the user manual.
- Keep this document and the user manual in a safe place. These documents are part of the product and must be readily available.

DANGER!

Danger of explosion

Do not use this device in an environment supplemented with the following gases:

- Oxygen
- Flammable anaesthetics
- Other flammable substances/air mixtures

CAUTION!

Patient hazard, damage to device

- Additional devices that are connected to medical electrical equipment must verifiably comply with the respective IEC or ISO standards (e.g. IEC 60950 for data processing equipment). Furthermore, all configurations must comply with the normative requirements for medical systems (see IEC 60601-1-1 or section 16 of the 3rd edition of IEC 60601-1, respectively). The person connecting additional devices to medical electrical equipment becomes the system configurer and is thereby responsible for ensuring the system's compliance with the normative requirements for systems.

It should be noted that local laws take priority over the above-mentioned standard requirements. For further information please contact your local specialist dealer or the Technical Service.

- Have the scales serviced and recalibrated regularly as described in the respective section in the user manual of the device.
- Technical changes to the device are not permitted. The unit contains no user-serviceable parts. Make sure that maintenance and repair are only carried out by an authorisedseca service partner. You can find your local service partner at www.seca.com or send an e-mail to service@seca.com.

- Make sure you only use genuine seca accessories and spare parts. Otherwise the warranty provided by seca will become null and void.

CAUTION!

Patient hazard, malfunction

- Make sure electrical medical equipment such as high-frequency surgical devices are kept at a minimum distance of approx. 1 metre to prevent incorrect measurements or interference with radio transmission.
- Make sure HF devices such as mobile phones are kept at a minimum distance of approx. 1 metre to prevent incorrect measurements or interference with radio transmission.
- The actual transmit power of HF devices can require minimum distances of more than 1 metre. For details, visit www.seca.com

Handling the chair scales

CAUTION!

Patient hazard, damage to device

- Use the chair scale only under the supervision of specialist staff.
- Prior to each use, ensure that the wheel brakes work properly.

- Secure the scales against accidental rolling. Lock the wheel brakes.
- Do not transport any persons or objects with the chair scales.
- Move the scales only with the wheel brakes released.

CAUTION!

Patients can be injured if they fall

People with limited motor skills can fall when they try to sit down on the scales.

- Ensure that the wheel brakes on the chair scales are set.
- Assist people with limited motor skills when they are sitting down.

- Leave the footrests folded up and pushed in until the patient has taken a seat on the scales.

CAUTION!

Injury due to trapping

Hands and arms of the patient and the user can become trapped between seat and base frame as well as between seat and arm-rests.

- Keep your arms and hands above the seat when you help the patient to sit down.
- Make sure that the patient's hands and arms are placed on the armrests or in his or her lap during the weighing process.
- Ensure that there are no objects between the seat and the base frame.
- Ensure that the armrests are folded down during the weighing process.
- Make sure that the patient's feet are placed on the footrests during the weighing process.

ATTENTION!**Incorrect measurement due to force shunts**

If accessories (e.g. power supply pouch) or pieces of clothing become trapped between the seat and the base frame, the weight will not be measured correctly.

If the patient's feet are not on the footrests, the weight measurement will be incorrect.

Handling the baby scales**WARNING!****Serious injury to baby due to a fall**

Baby scales generally stand on raised work surfaces. If the baby falls from this surface, this may result in serious irreversible or fatal injuries.

- Never leave a baby unsupervised on the scales.

Preventing electric shock**WARNING!****Electric shock**

- Set up the device so that the power supply socket is easy to reach and the device can be disconnected from the power supply quickly.
- Ensure that your local power supply matches the information on the power supply unit.
- Do not touch the power supply unit with wet hands.
- Do not use extension cables or power strips.
- Ensure that the power cable is not pinched or damaged by a sharp edge.
- Do not operate the device above an altitude of 3000 m.

Avoiding infections

WARNING!

Risk of infection

- Hygienically reprocess the scales regularly as described in the respective section in this document.

- Make sure that the patient has no infectious diseases.
- Make sure that the patient has no open wounds or infectious skin alterations, which may come into contact with the device.

Avoiding injury

WARNING!

Risk of falling

- Ensure that the device is positioned firmly and level.
- Route the mains cable so that there is no risk of tripping for the patient or user.
- The device is not designed as a standing aid. Assist people with limited motor skills when they are getting up, e.g. from a wheelchair.
- Make sure that the patient does not step onto the weighing platform directly at the edges.

- Make sure that the patient steps onto the weighing platform slowly and securely.

WARNING!

Danger of slipping

- Ensure that the weighing platform is dry before the patient steps onto it.
- Ensure that the patients feet are dry before he or she steps onto the weighing platform.
- Make sure that the patient steps onto the weighing platform slowly and securely.

Avoiding damage of the device

ATTENTION!

Damage to device

- Ensure that no liquids enter the device. They can damage the electronics.
- Switch off the device before disconnecting the power supply unit from the mains socket.
- Disconnect the power supply unit from the mains socket if you intend to not use the device for a longer period of time. Only this way it can be ensured that the device is currentless.

- Make sure not to drop the device.
- Do not expose the device to any impacts or vibrations.
- Perform function controls regularly as described in the relevant section in this document. Do not operate the device if it is damaged or not working properly.
- Ensure that there is no heat source in the immediate vicinity. Do not expose to direct sunlight. The excessive temperature could damage the electronics.

- Avoid rapid temperature fluctuations. When the device is transported so that a temperature difference of more than 20 °C occurs, it must stay turned off for at least 2 hours before it can be

turned on again. Otherwise, condensation water will form which can damage the electronics.

- Caustic detergents may damage the surfaces. Only use a soft cloth dampened with mild soap-suds to clean the surfaces of the device.

Handling the measurement results

WARNING!

Patient hazard

This device is **no** diagnostic device. It simply assists the treating physician in establishing a diagnosis.

- In order to make a precise diagnosis and initiate therapeutic measures, besides determination of the weight, further targeted examinations must be set up by the physician, and their results must be considered.
- The responsibility for diagnosis and treatment lies with the treating physician.

CAUTION!

Patient hazard

In order to avoid misinterpretations, test results for medical use must be displayed and used in SI units only. Some devices offer the ability to display test results in other units. This is only an additional function.

- Use the results exclusively in SI units.
- The use of measurement results in non-SI units is the sole responsibility of the user.

ATTENTION!

Loss of data

- Before you save measurements acquired with this device and process them for further use (e.g. in the PC software **seca analytics** or in a hospital information system), make sure the measurement values are plausible.
- If measurement values are transmitted to the **seca analytics** PC software or a hospital information system, make sure prior to further use that the measurement values are plausible and are assigned to the correct patient.

Handling the packaging material

WARNING!

Danger of suffocation

Packaging material made of plastic foil (bags) is a choking hazard.

- Keep packaging material out of reach of children.
- In the event that the original packing material may not be available anymore, only use

plastic bags with security holes in order to reduce the risk of suffocation.

NOTE:

Keep the original packing material for future use (e.g. returning for maintenance service).

3.3 Handling the (rechargeable) batteries

WARNING!

Personal injury due to improper handling.

Batteries contain harmful substances which may explode if not handled properly.

- Do not try to recharge batteries.
- Do not expose (rechargeable) batteries to heat.
- Do not burn (rechargeable) batteries.
- If acid is leaking out, avoid contact with the skin, eyes and mucous membranes. Rinse affected areas with plenty of clean water and seek medical help at once.

ATTENTION!


Damage to device and malfunctions with improper handling

- Only use the type of (rechargeable) battery specified in this document.
- When replacing (rechargeable) batteries, always replace a complete set at a time.
- Do not short-circuit (rechargeable) batteries.
- If you do not use the device for a long period of time, remove the batteries (incl. rechargeable batteries). This prevents acid from leaking into the device.
- If acid leaked into the device, discontinue use. Have the device checked by an authorised secaservice partner and repaired if necessary.

4. OVERVIEW

The controls, symbols and signs described in this section may be combined differently depending on the device type. For details about the actual functionalities of your seca device, please see the respective manual.

4.1 Control elements

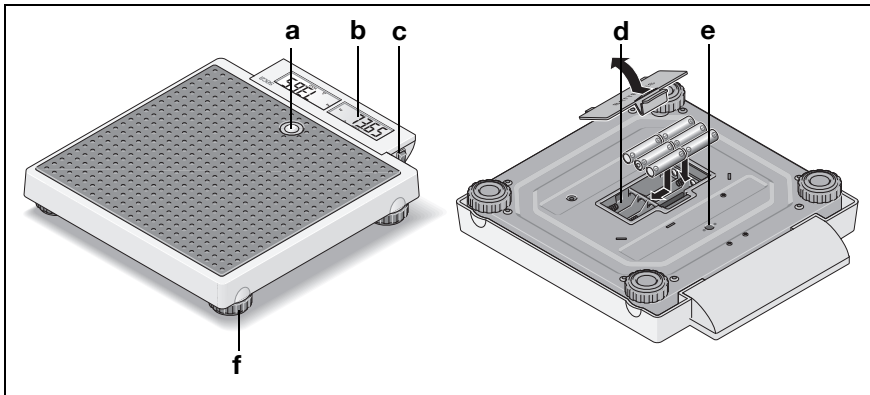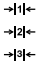
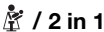
No.	Control element	Function
Standard control elements		
a	Spirit level	Indicates whether the device is horizontal
b	Display	Display element for depiction of measurement results and for device configuration
c	start / 	On/Off button; On/Off switch - To switch scales on and off
d	Battery compartment	Compartment for batteries or rechargeable battery block
e	Socket for power supply unit	For connection of power supply unit
f	Foot screw	For precise alignment of device

Control element	Function
Control elements dependent on model	
	Arrow key <ul style="list-style-type: none"> • In the menu: <ul style="list-style-type: none"> - Select submenu, select menu item - Increase value

Control element	Function
▼	Arrow key <ul style="list-style-type: none"> • In the menu: <ul style="list-style-type: none"> - Select submenu, select menu item - Reduce value
↶	Enter key <ul style="list-style-type: none"> • During weighing (if radio network is set up): <ul style="list-style-type: none"> - Press briefly: Send measured result to receive-ready equipment (PC with USB wireless module) - Hold down: Print out measured result (wireless printer) • In the menu: <ul style="list-style-type: none"> - Confirm selected menu item - Save set value
hold	hold key <ul style="list-style-type: none"> - Activate Hold function, display measured results continuously
tare	tare key <ul style="list-style-type: none"> - Reset display to zero; activate/deactivate Tare
pretare	pretare key <ul style="list-style-type: none"> - Activate/deactivate Pre-Tare
BMI	BMI key <ul style="list-style-type: none"> - Activate BMI function
2 in 1	2 in 1 key <ul style="list-style-type: none"> - Start the 2 in 1 function for weighing babies and toddlers
Func	Func key <ul style="list-style-type: none"> - Call-up other functions (e.g. BMI, Pre-Tare, ...)
send	send key <ul style="list-style-type: none"> • During weighing (if radio network is set up): <ul style="list-style-type: none"> - Send measured result to receive-ready devices (wireless printer, PC with USB wireless module)
print	print key <ul style="list-style-type: none"> • During weighing (if radio network is set up): <ul style="list-style-type: none"> - Print out measured result (wireless printer)
input	input key <ul style="list-style-type: none"> • During weighing: <ul style="list-style-type: none"> - Enter patient data (age, gender, PAL)
menu	menu key: <ul style="list-style-type: none"> • During weighing: <ul style="list-style-type: none"> - Call up control panel menu • In the menu: <ul style="list-style-type: none"> - Press briefly: Go back one menu level - Hold down: Exit menu


Control element	Function
clear	clear key: - Clear manually entered data or data received wirelessly (patient data, body length, BMI, BFR)
gender	gender key - Enter gender of patient
kg/lbs	kg/lbs key (not in calibrated models): - Switch weight unit
g/oz	g/oz key (not in calibrated models): - Switch weight unit
DATA VALID	DATA VALID key - Transmit current length value to scales
ZERO	ZERO key - Set stadiometer to zero at current position


4.2 Symbols in display

Symbol	Meaning
	Battery power low
	Operation with power supply unit
	Non-calibratable function active
	Weighing range currently in use
BMI	Body Mass Index is displayed
BFR	Body Fat Rate is displayed
PT	Pre-Tare function is activated
TARE	Tare function is activated
NET	Tare function is activated
HOLD	Hold function is activated
	Mother-child function (2 in 1 function) for weighing babies and toddlers is active
	Differential value function (BMIF : Breast Milk Intake Function) active
	Measured result in memory location 1 (BMIF function)
	Select memory location (BMIF function)
	Differential function is activated
M	Displayed when differential display function is activated, and the weight value is stored
kg / g	Weight in kilogrammes/grammes
lb / lbs	Weight in pounds (not in calibrated models):

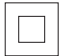Symbol	Meaning
oz	Weight in ounces (not in calibrated models):
sts	Weight in stones (not in calibrated models):
cm	Body height in centimetres
ins	Body height in inches (not in calibrated models)
%	Unit for display of body fat rate
kg/m²	Unit for display of body mass index
	Key sound is activated
	Device is equipped with a radio module; data are received and transmitted wirelessly
♀ / ♂	Patient's gender
	Arrow points to current indication/function
zero	Set zero point (in devices with integrated height measurement); all heights are measured in relation to this zero point. If the zero point is undershot, the measured values are shown with a minus sign in front of them.

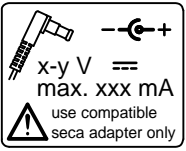4.3 Information on device and on rating plate

Text/symbol	Meaning
Model	Model number
Ser. No.	Serial number
	Refer to user manual
	Observe user manual
	Type B electromedical device
	Type BF electromedical device
	Protection class II device with protective insulation
M	Conformity label according to Directive 2009/23/EC governing non-automatic weighing instruments (calibrated models)
e	Value in mass units, which is used for classification and for calibration of scales (calibrated model)

Text/symbol	Meaning
d	For electronic scales: Value indicating the difference between two successive measurement values For mechanical scales: Value in mass unit, which indicates the difference between the values of two adjacent increments
→ ←	Weighing range (calibrated models)
	Scales of calibration class III according to Directive 2009/23/EC
	Scales of calibration class IIII according to Directive 2009/23/EC
	Device complies with EC standards and directives. <ul style="list-style-type: none"> • xx: year in which EC-calibration was carried out and the CE-label was placed (calibrated model). • 0109: Notified body metrology: Office of Legal Metrology of the state of Hessen (Germany) (calibrated models) • 0123: Notified body medical products: TÜV Süd Product Service
	Symbol of US authority FCC (Federal Communications Commission)
FCC ID	Device licensing number issued by US FCC (Federal Communications Commission) authority
IC	Device licensing number issued by the Industry Canada authority
	Rating plate on power supply unit socket <ul style="list-style-type: none"> • x-y V: required power supply • max xx mA: maximum power consumption • --⎓+ : Note the polarity of the connector plug • --- : Operate device with continuous current (DC)
	Do not dispose of device in household waste

4.4 Information on the packaging

	Protect from moisture
	Arrows indicate top of product. Transport and store in an upright position.

	Fragile Do not throw or drop.
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. moisture for transport and storage
	Packaging material can be disposed of through recycling programs.


5. BEFORE YOU GET STARTED ...

5.1 Wall mounting of cabled remote display

Various models are equipped with a cabled remote display. Choose an adequately strong, load-bearing wall to mount the device. Standard rawl plugs are supplied for installation on solid brick walls. We recommend use of appropriate special rawl plugs for other types of walls.

Installing the wall brackets

In order to install the wall brackets, proceed as follows: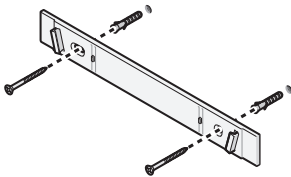


1. Mark the drillholes at the desired height.

NOTE:

Consider the cable length of the respective cabled remote displays when determining the height.

2. Mark the drillholes for the wall brackets perpendicular to each other.
3. Drill the holes using a drill bit \varnothing 5 mm.
4. Insert the rawl plugs in the drill holes.
5. Mount the wall brackets using the screws.

6. Fit the cabled remote display on the wall brackets.

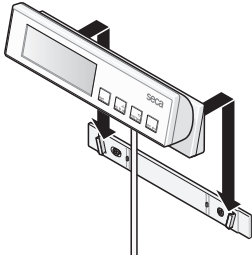ATTENTION!

Incorrect measurement due to force shunts

If the cable of the cabled remote display is touching the weighing platform, the weight measurement will be incorrect.

- Route the cable in such a way that it cannot touch the weighing platform.

5.2 Establish power supply

Connect power supply unit

WARNING!

Personal injury and damage to device due to use of incorrect power supply unit

The voltage supplied by standard power supply units may be higher than their indicated rating. This may cause the scales to overheat, catch fire, melt or short-circuit.

- Only use genuine seca plug-in power supply units with 9V or a controlled 12 V output voltage.

Charge battery

Various models are equipped with an E-block rechargeable battery. The battery is charged automatically if the power supply is established via plug-in power supply unit. Leave the product connected to the power supply for at least 24 hours during operation in order to fully charge the battery block.

6. HYGIENIC REPROCESSING

WARNING!

Electric shock

Electronicseca devices are not currentless when the power button is pressed and the display is turned off. The application of liquids on the device may cause an electric shock.

- In order to make the device currentless, disconnect the power supply unit from the mains prior to each hygienic reprocessing.
- Ensure that no liquids enter the device.

CAUTION!

Damage to device

Inappropriate cleaning detergents may damage the sensitive surfaces of the devices.

- Do not use caustic or abrasive detergents.
- Do not use spirits or benzine.

6.1 Cleaning

- ◆ Use a soft cloth dampened with mild soapsuds to clean the surfaces of the device.

6.2 Desinfection

1. Follow the instructions on the disinfectant.
2. Disinfect the device at regular intervals with a commercially available disinfectant, e.g. with 70% iso-propyl alcohol.

Please take note of the following terms:

- Prior to every measurement with direct skin contact:
 - Weighing platform
- After every measurement with direct skin contact:
 - Weighing platform
- If required:
 - Display
 - Membrane keyboard

6.3 Sterilisation

The sterilisation of this device is not permitted.

7. FUNCTIONAL CHECK

- ◆ Perform functional check prior to each operation.

A complete functional check includes:

- Visual inspection for mechanical damage
- Visual inspection of the display
- Functional check of the control elements, as applicable:
 - Fixing brakes
 - Transport rollers
 - Keys
 - Armrests
 - Footrests
 - Standing aid
 - Fold-up seat
 - Foot screws
- Functional check of optional accessories

If you notice any faults or deviations during the functional check, try to resolve the error by consulting the user manual.

CAUTION!

Damage to persons

If you notice any faults or deviations during the functional check, try to resolve the error by consulting the user manual.

- Have repairs of the unit carried out by seca service or an authorized dealer.

8. WHAT DO I DO IF ...

<p>... no weight is displayed during weighing?</p>	<p>The scales are not connected to the power supply.</p> <ul style="list-style-type: none"> - Check if the scales are switched on. - Check if battery block is inserted.
<p>... "0.00" does not appear before weighing?</p>	<p>A load was placed on the scales before being switched on.</p> <ul style="list-style-type: none"> - Remove load - Switch scales off and back on again
<p>... a segment lights up continuously or not at all?</p>	<p>The relevant segment is faulty. Please notify maintenance service.</p>
<p>...  is displayed?</p>	<p>Voltage of batteries or battery block is dropping.</p> <ul style="list-style-type: none"> - Recharge battery block as soon as possible. - Replace batteries as soon as possible.
<p>... the display "StOP" appears?</p>	<p>The maximum load has been exceeded.</p> <ul style="list-style-type: none"> - Remove load.
<p>... the display "bAtt" appears?</p>	<p>Batteries or battery block are flat.</p> <ul style="list-style-type: none"> - Charge battery block. - Replace batteries.
<p>... the display "tEMP" appears?</p>	<p>The ambient temperature of the scales is too high or too low.</p> <ul style="list-style-type: none"> - Set up scales in an ambient temperature between +10 °C and +40 °C. - Wait approx. 15 minutes until scales have adjusted to ambient temperature.
<p>... the display "Er" or "EEEE" appears (depending on the model)?</p>	<p>The automatic software self-test of the scales has found an error.</p> <ul style="list-style-type: none"> - Restart scales. - If error message appears again, please contact maintenance service.
<p>... the display "Er:X:11" appears?</p>	<p>Excessive weight has been applied in general or on one edge.</p> <ul style="list-style-type: none"> - Reduce load on scales or distribute weight more evenly. - Restart scales.
<p>... the display "Er:X:12" appears?</p>	<p>The scales have been switched on with too great a load.</p> <ul style="list-style-type: none"> - Remove load. - Restart scales.
<p>... the display "Er:X:16" appears?</p>	<p>Oscillations have been induced in the scales, preventing determination of the zero point.</p> <ul style="list-style-type: none"> - Restart scales.

9. TECHNICAL DATA

Technical data	
Temperature range <ul style="list-style-type: none"> • Operation • Storage • Transport 	+10 °C to +40 °C (50 °F to 104 °F) -10 °C to +65 °C (14 °F to 149 °F) -10 °C to +65 °C (14 °F to 149 °F)
Air pressure <ul style="list-style-type: none"> • Operation • Storage • Transport 	700 - 1060 hPa 700 - 1060 hPa 700 - 1060 hPa
Humidity <ul style="list-style-type: none"> • Operation • Storage • Transport 	30% - 80% not condensing 0% - 95% not condensing 0% - 95% not condensing
EN 60601-1: <ul style="list-style-type: none"> • Protection class II device with protective insulation depending on model <ul style="list-style-type: none"> • Type B electromedical device • Type BF electromedical device 	  
Type of protection	IP20
Duty cycle	Continuous duty

10. OPTIONAL ACCESSORIES

Model designation	Article number
Ramp for multifunctional scales	470-00-00-009
Carrying bag 421 for seca 869, 878 and similar models	421-00-00-009
RS232 adapter kit with printer interface for seca 763, 764	460-00-00-009
Mechanical stadiometers for column scales <ul style="list-style-type: none"> • seca 220, metric • seca 220, metric, imperial 	220-17-14-004 228-18-14-009
Mechanical stadiometers for multifunctional scales <ul style="list-style-type: none"> • seca 223, metric • seca 223, metric, imperial 	223-17-17-999 223-18-14-999
Mechanical stadiometers for baby scales <ul style="list-style-type: none"> • seca 231, metric • seca 231, metric, imperial • seca 232, metric 	231-17-17-009 231-18-14-009 232-17-17-009

11. OPTIONAL ACCESSORIES FOR SECA 360° WIRELESS NETWORK

seca 360° wireless Devices	Article number
Length measuring device <ul style="list-style-type: none"> • seca 274 • seca 264 	Country-specific variants Country-specific variants
Wireless printer <ul style="list-style-type: none"> • seca 360° Wireless Printer 465 • seca 360° Wireless Printer Advanced 466 	Country-specific variants Country-specific variants
PC software <ul style="list-style-type: none"> • seca analytics 105 • seca analytics 115 	Application-specific licence packages Application-specific licence packages
seca 360° Wireless USB adapter 456	456-00-00-009

12. SPARE PARTS

Spare parts	Article number
Power supply unit 230 V~/50 Hz/12 V=/130 mA	68-32-10-252
Switch-mode power supply unit: 100-240V~/ 50-60Hz / 12V= / 0.5A	68-32-10-265
Switch-mode power supply unit: 100-240V~/ 50-60Hz / 12V= / 0.5A with safety coupling	68-32-10-266
Recalibration sticker	14-05-01-886
Battery block	68-22-12-721

13. DISPOSAL

13.1 Device

Do not dispose of the device with household waste. The device must be disposed of properly as electronic waste. Comply with the national provisions applicable in your country. For further information contact our service department at:

service@seca.com

13.2 Batteries (including rechargeables)

Spent (rechargeable) batteries should not be discarded with household waste, regardless of whether they contain harmful substances or not. As a consumer you are obliged by law to dispose of (rechargeable) batteries via the collection points set up by the municipal authorities or the retail sector. Only discard (rechargeable) batteries when fully discharged.

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