



MICROSAFE® Tube

Principle: An air vent is positioned on the sidewall of the tube to provide 1) automatic sample filling, 2) precise control of the sample volume and 3) sample dispensing. **For good operation, blood sample MUST fill the air vent at the end of the sampling, up to the line mark.**

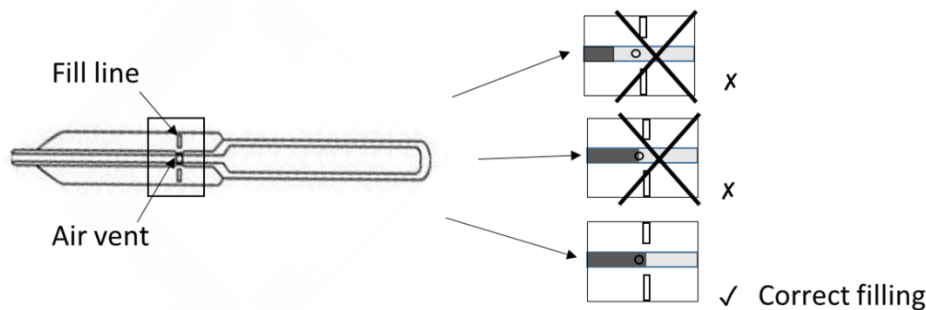

Step 1: Hold the tube horizontally, and touch the open end of the tube to the blood sample. The blood sample will fill automatically by capillary action.

Keep the tube touching the blood sample until it fills completely to the air vent, at the level of the fill line. Filling will stop automatically when the blood sample clogs the air vent.

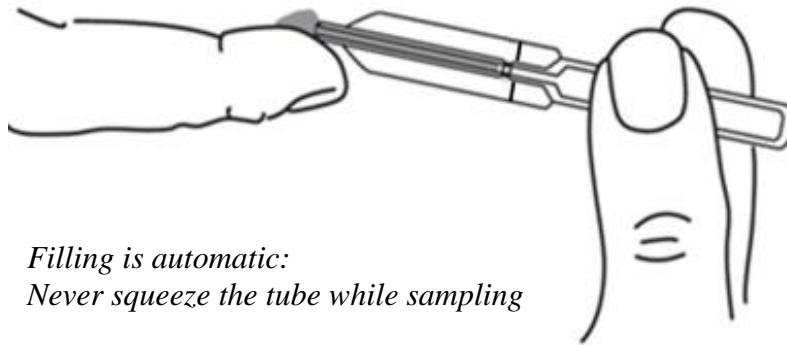

CAUTION!

*Filling is automatic:
Never squeeze the tube while sampling*


Step 2: To dispense the blood sample, align the open end of the tube with the sample location of the consumable and gently squeeze the bulb of the tube, to prevent air bubbles, until the sample is completely dispensed.

CAUTION!


Allow the tube to fill all the way. If a sample do not expel, carry on sampling blood until the air vent filling.

MICROSAFE® Tube

Principe : Un évent (trou d'aération) est percé sur la paroi latérale du capillaire pour permettre 1) le remplissage automatique de l'échantillon, 2) le contrôle précis de son volume et 3) le dépôt de l'échantillon. **Il est IMPERATIF de remplir la pipette jusqu'à remplir l'évent au niveau de la ligne.**

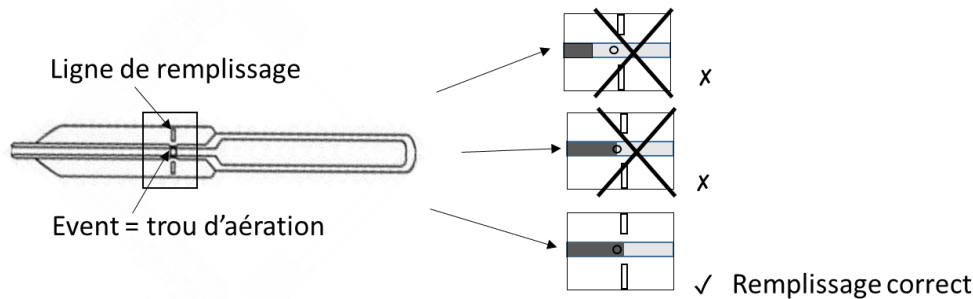

Step 1: Tenez le capillaire horizontalement et touchez l'échantillon de sang avec la pointe du capillaire. L'échantillon de sang montera automatiquement par capillarité. Maintenez le tube en contact avec l'échantillon de sang jusqu'à ce qu'il se remplisse complètement jusqu'à l'évent, au niveau de la ligne de remplissage. Le remplissage s'arrêtera automatiquement lorsque l'échantillon de sang bouchera l'évent.

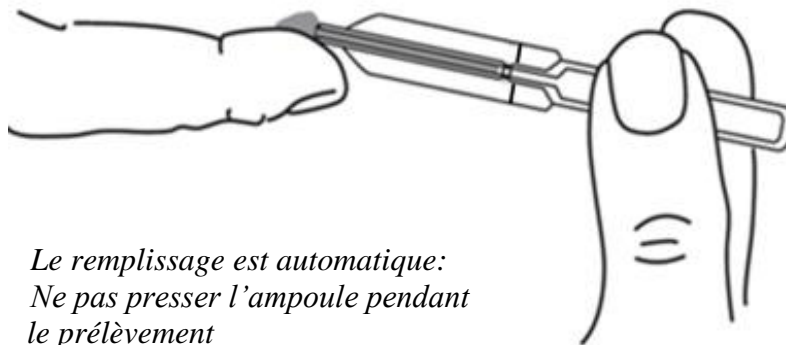

ATTENTION!

*Le remplissage est automatique:
Ne pas presser l'ampoule pendant
le prélèvement*

Step 2: Pour déposer l'échantillon de sang, alignez l'embout du capillaire avec la zone de dépôt du consommable et presser délicatement l'ampoule, en évitant les bulles, jusqu'à ce que l'échantillon soit complètement distribué.

ATTENTION!

Laisser le capillaire se remplir jusqu'au bout. Si un échantillon n'est pas expulsé, prélever à nouveau l'échantillon jusqu'à remplir l'évent.